



SCHWERT
SWORD · ESPADA
EPEE · SPADA

SAFE-EX
DBGM

1033-60 Care and Maintenance

Maintenance and re-processing of SAFE-EX Blade-Holder

DISMANTLING:

The SAFE-EX Blade Holder has to be dismantled before cleaning, as shown.

Attention: Don't remove the spring.

CLEANING:

Please consider our "Application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664:2004"

Pre-Treatment: Use cleaning brush with nylon bristles when incrustations are heavy.

Cleaning: Manual or mechanical – **Attention:** Cleaning in an thermal disinfectant: standing vertically, thread downwards.

Rinsing: Please use deionized water to avoid water spots.

Drying: Use compressed air or air-dry (manual preparation).

MAINTENANCE:

Oiling with Medical White Oil, as shown.

ASSEMBLY:


Assembly is made in reverse order to the dismantling.




3297 50 ml
Paraffinum Perliquidum
(Paraffin Oil / Medical White Oil)

For maintenance of instrument hinges and for use as sharpening oil. Physically harmless to DAB 10.

1 Turn off the back end by twisting the handle counterclockwise

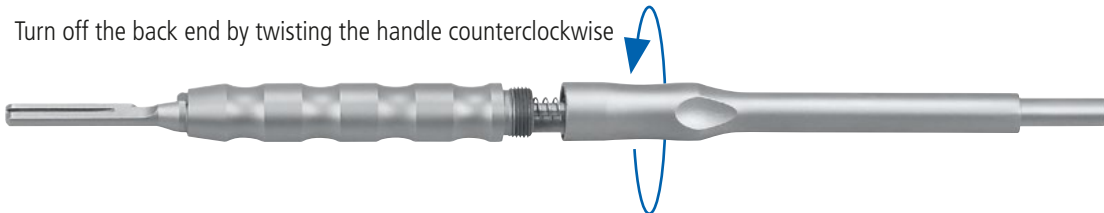

2 Remove the back end and the insert

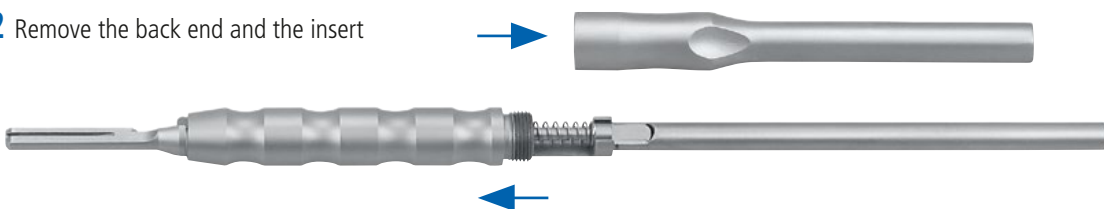

3 Oiling after cleaning with Medical White Oil

