



Product insights  
you can trust.

# OMNICHROMA

## EDITORS' CHOICE

++++ 1/2

**RATING SYSTEM:** Excellent + + + + + Very Good + + + + Good + + +



**Tokuyama**

www.tokuyama-us.com

### Description

**OMNICHROMA** is a universal, radiopaque, one-shade, resin-based composite. It incorporates Smart Chromatic Technology, a unique technology based on fillers that generate red-to-yellow structural color. As a result, the cured composite blends with the surrounding tooth structure of almost any patient. The composite contains uniform supra-nano spherical fillers and round fillers, fabricated with silicon dioxide and zirconium dioxide. Prior to light curing, the paste is opaque-white and transforms upon curing to blend with the surrounding tooth structure. For large Class III and IV restorations, a thin layer of **OMNICHROMA BLOCKER** can be applied on the lingual aspect of the preparation to prevent shade matching interference from the oral cavity. It can also be used to mask staining or to restore a highly opaque tooth, maintaining the ability of the composite to match the surrounding tooth structure. **OMNICHROMA** is available in syringes and pre-loaded tips (PLTs).

### Unique Features

- Smart Chromatic Technology, with supra-nano spherical fillers and round fillers
- One shade matches all tooth shades from A1 to D4
- High compressive and flexural strength
- High polishability and luster
- Low polymerization shrinkage
- Wear- and acid-resistant
- **OMNICHROMA BLOCKER** masks stains and the oral cavity in large Class III and IV preparations

"ONE AND DONE. BLENDS IN A WIDE VARIETY OF SHADES AND ELIMINATES THE NEED TO CHOOSE ONE."

### Consultants' Comments

- "The transition from opaque-white to tooth shade blending after curing is amazing."
- "Creamy consistency, stacks well and handles beautifully."
- "Easy to pack. Did not stick to instruments or pull away from the tooth during placement."
- "90% of the time, a perfect shade. Other times, the value was neutral or slightly translucent."
- "This concept could be a real game changer for everyday dental restorations."
- "Not well suited for extreme white (bleach) shades."
- "Somewhat opaque after curing, and it needs to be more radiopaque."

94%  
overall  
rating

### Indications

- Anterior and posterior direct composite restorations
- Composite and porcelain repairs
- Direct, resin-bonded veneers
- Diastema closures

### Clinical Tips


- Shield the material from the headlamp to prevent premature curing.
- Proper bonding procedure is vital for shade matching to work correctly.
- Incisal edges are tricky. This product is excellent in certain areas, not so much in others.
- The best product to fill in crowns with endo holes. The **BLOCKER** masks the metal and the resin matches nicely.

### Evaluation Highlights


**OMNICHROMA** was evaluated by 36 consultants and used 757 times in total.

- One-shade composite that cures to match all shades from A1 to D4
- No need to match shades, saving time
- **BLOCKER** masks stains and blocks out the oral cavity
- Wear-resistant and acid-resistant

#### Key Features:

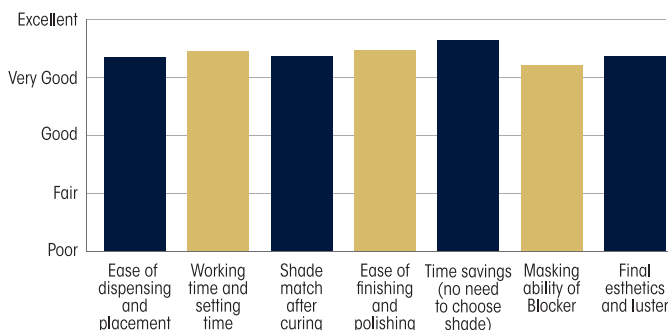

#### Compared to Competitive Products:

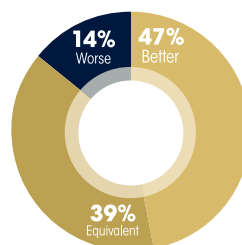

#### Percentage of Consultants Who Would:

